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European Medical Device Regulation
This product complies with Regulation 2017/745 on medical devices. 

1 Intended Purpose

Electrosurgical equipment for tissue cutting and coagulation.

1.1 Indications

Single-use neutral electrodes are self-adhesive, ready-to-use, single-use products and are accessories for monopolar applications in HF surgery. The electrodes are designed to close the electrical circuit between the HF generator and the patient on the passive side.

1.2 Contra-indications

The neutral electrode must not be used on damaged or injured skin.
Applying the neutral electrode to areas of skin with tattoos can cause skin damage as tattoo ink may contain electrically conductive components.
Neutral electrodes must not be used in direct contact with the heart, central circulatory system or central nervous system.
Do not use neutral electrodes if their surgical techniques are contraindicated.
Do not use neutral electrodes if, in the opinion of an experienced physician or according to current professional literature, such use would create a hazard to the patient, e.g. due to the patient's general condition, or others.

2 Physical principle of operation

The neutral electrodes are designed to close the electrical circuit between the HF generator and the patient on the passive side.

3 Safety instructions

3.1 Device-related

- Follow the operating instructions noted on the neutral electrode packaging
- Do not use this product after the expiry date
- Only use neutral electrodes that are compatible with your HF generator.
Please refer to the compatibility list (please contact the manufacturer)
- Neutral electrodes in an opened packaging bag have a maximum shelf life of 7 days
- When adjusting the HF generator settings be aware of any power limitations of the neutral electrode.
- Follow the instructions for use of the HF generator and the associated connection cable
- The "BF"/"CF" applied part of the HF unit used is extended by the instrument connected to it
- Any modification of the neutral electrode (e.g. shortening) will void the manufacturer's liability.

3.2 Use-related

- The neutral electrodes may only be used by trained medical professionals
- Users must be trained in, and be familiar with, the basic principles & rules of the application, and risks of HF surgery.
- All serious incidents arising in connection with the product must be reported to the manufacturer and the competent authority of the Member State in which the user is established
- Verify that the neutral electrode is in perfect working order before using it.
- Make sure that no short circuit can occur at the neutral electrode connection
- The neutral electrodes are ready to use:
 - Do not use any additional contact gel
 - The neutral electrodes are not sterile products and must not be sterilised.

3.3 EMC information

Medical electrical devices are subject to special EMC precautions, so please follow the instructions detailed below.

- BOWA accessories are only intended for connection to the specified HF devices.
- Using the accessory with medical devices from other manufacturers may result in higher emission levels or reduced interference immunity.

NOTICE

Combining medical devices is safe only if

- the relevant instructions for use permit the desired combination or
- the intended use and the interface specification of the products used in the combination are compatible.

The Instructions for use and interface specifications of medical devices used in combination must be strictly observed.

4 Procedure during use

4.1 Electrode selection

Note the following areas of application for the neutral electrode:

Pictogram	Area of application	REF without cable	REF with cable
	Universal electrodes for patients with a body weight of > 5 kg (adults and children)	816-072 816-092 816-112 815-110	818-072 818-092 818-112 817-110
	Electrodes for patients with a body weight > 15 kg (adults)	816-162 816-161 816-141 815-140	818-162 818-161 818-141 817-140
	Electrodes for patients with a body weight of 5 – 15 kg (children)	816-071 815-070	818-071 817-070
	Electrodes for patients with a body weight < 5 kg (newborns)	816-042 815-040	818-042 817-040

Split neutral electrodes

If your HF device has a contact quality monitoring circuit (CQM, e.g.: "EASY", "REM", "NESSY"), the use of split neutral electrodes is always recommended. The contact to the patient is monitored automatically when split neutral electrodes are used with a CQM system.

Non-split neutral electrodes

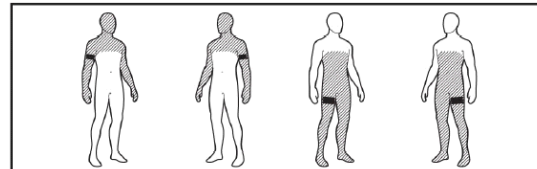
The contact to the patient is not monitored if used with HF devices that do not incorporate a CQM system or when non-split neutral electrodes are used.

Regularly check that the neutral electrode is in complete contact with the skin, especially whenever the patient is re-positioned.

A partially detached neutral electrode can damage the skin

4.2 Application notes

Suitable application site:

- Select the application site for the neutral electrode appropriately such that the current paths between the active and neutral electrodes are as short as possible and run longitudinal or diagonal to the body, as the conductivity of muscles is higher in the direction of the fibrils
- Depending on the location of the surgical site, preferably apply the neutral electrode to the nearest upper arm or thigh, always ensuring a minimum distance of 20 cm between electrode and surgical site.
- The neutral electrode must be positioned on the upper side of the body on a reclining patient in order to prevent it from being compromised
- The application site must be dry. If applicable, allow cleaning agents to dry completely

Preparing the patient:

- The neutral electrode must not be used on damaged or injured skin
- Remove any metal jewellery
- Remove thick hair growth
- Clean the application site without using alcohol as this dries out the skin and increases the contact resistance
- In cases of poor circulation, massage or brush the application site

Preparing the electrode:

- The neutral electrode must be visually inspected for damage before use.
If damaged, the neutral electrode must not be used
- Remove the protective film

Application:

- Apply the neutral electrode
- Make sure that the longer edge of the neutral electrode has full skin contact with the surgical site.
This prevents an increase in current density at the short edge
Other types of neutral electrodes, e.g. REF 816-092, 818-092, 816-072, 818-072 are direction-independent and freely orientable due to their design
- Avoid applying the electrode to thick subcutaneous fat tissue
- Make sure that the application site is free of scar tissue, tattoos, bone spurs and thick hair growth
- Make sure that there are no ECG electrodes or implants (e.g. bone pins, bone plates, endoprotheses) in the current path
- Avoid sites at which fluids can accumulate
- Avoid positioning the neutral electrode under the patient's buttocks or back
- Operating on the thorax, do not run the current path transversely across the patient's body and make sure that the heart is never in the current path

Contacts:

- Apply the neutral electrode firmly to the patient's body
- Make sure that the neutral electrode is applied fully and without any wrinkles in order to prevent air bubbles from being formed under the neutral electrode
- Make sure that the protective edge adheres to the patient completely and without any wrinkles

Placing the connection cable:

- Do not guide or coil the cable around an arm, leg or metal object
- Do not run the cable over or under the patient
- Do not position any cable terminals under the patient

Removing the neutral electrode:

- Always disconnect the electrode connecting cable at the plug
- Remove the neutral electrode carefully after use, beginning at a corner
- Pulling the neutral electrode off too quickly can cause skin irritation (redness)

Single use only:

- Neutral electrodes are designed for single use only
- Do not reapply a neutral electrode after applying it incorrectly
- Re-use poses a risk of HF burns to the patient as the neutral electrode's essential performance features may be compromised

5 Ambient conditions

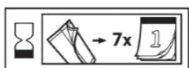
5.1 Transport and storage

- Open the packaging immediately before use
- Neutral electrodes in an opened packaging bag have a maximum shelf life of 7 days
- Store the neutral electrode, in its original packaging, in a dry location
- Avoid direct sunlight and proximity to heat sources
- The permissible storage temperature is in the range of 0°C to 40°C

5.2 Operation

Use the product at room temperature.

6 Disposal

Dispose of the medical devices, packaging material and accessories in accordance with the respective applicable country-specific regulations and laws.

7 Symbols to note

Symbol	Definition
	Follow the instructions for use
REF	Catalogue number
LOT	Batch code
QTY	Quantity
	Keep away from direct sunlight
	Keep dry
	Manufacturer
CE ₀₁₂₃	CE mark and identification number of notified body
UK CA	Approval mark United Kingdom
Rx only	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician
MD	Medical device
	Temperature limit
	Single use only
	Use-by date
	Follow the instructions for use or electronic instructions for use
	Latex-free

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