DEFENDO™

Single Use Valve Kits

INSTRUCTIONS FOR USE

For OLYMPUS® and FUJIFILM™ GI Endoscopes

Reorder No. 100301

Single Use Biopsy Valve

Reorder No. 100303

Single Use Y-OPSY™ Valve

For PENTAX® GI Endoscopes

Reorder No. 100302

Single Use Biopsy Valve

For OLYMPUS® GI Endoscopes

Reorder No. 100305

Single Use Air/Water, Suction, and Biopsy Valve

Reorder No. 100306

Single Use Air/Water and Suction Valve

Reorder No. 100310

Single Use Air/Water, Suction, and Biopsy Valve and ENDOGATOR™ Connector

Reorder No. 100311

Single Use Air/Water Valve, Suction Valve, and ENDOGATOR™ Connector

Reorder No. 100322

Single Use Air/Water Valve, Suction Valve, Biopsy Valve, ENDOGATOR™ Connector and Cleaning Adapter

Reorder No. 100323

Single Use Air/Water Valve, Suction Valve, Biopsy Valve and Cleaning Adapter

For FUJIFILM™ 500/600 Series GI Endoscopes

Reorder No. 100312

Single Use Air/Water, Suction, and Biopsy Valve and ENDOGATOR™ Connector

Reorder No. 100313

Single Use Air/Water, Suction, and Biopsy Valve

Reorder No. 100314

Single Use Air/Water and Suction Valve

For PENTAX® 90 Series GI Endoscopes

Reorder No. 100315

Single Use Air/Water, Suction, and Biopsy Valve and ENDOGATOR™ Connector

Reorder No. 100316

Single Use Air/Water, Suction, and Biopsy Valve

Reorder No. 100317

Single Use Air/Water and Suction Valve

Intended Use:

The Single Use Biopsy Valve is intended for covering the endoscope biopsy port during an endoscopy procedure. The Single Use Biopsy Valve provides access for endoscopic device passage and exchange, helps maintain insufflation, and minimizes leakage of biomaterial from the biopsy port throughout the endoscopic procedure.

The Single Use Y-OPSY™ Biopsy Valve Irrigator is intended to enable simultaneous irrigation and instrumentation in the endoscope's biopsy channel.

The Single Use Air/Water Valve is intended to be used to control the air/water function on an endoscope during a GI endoscopic procedure.

The Single Use Suction Valve is intended to be used to control the suction function on an endoscope during a GI endoscopic procedure.

The Single Use ENDOGATOR™ Connector is intended to be used in conjunction with ENDOGATOR™ Irrigation Tubing to provide irrigation via sterile water during GI endoscopic procedures when used with an irrigation pump (or cautery unit).

The Single Use Cleaning Adapter is intended to be used only to pre-clean an endoscope's air/water channel, and not to be used during a patient procedure.

Warnings and Precautions:

- Guidelines for Preventing Cross Contamination:
 - DO NOT RESTERILIZE OR REPROCESS.
- The appropriate Single Use Biopsy Valves, Y-OPSY Biopsy Valve Irrigators, Air/Water Valves, and Suction Valves and Cleaning Adapters are to be used with the compatible endoscope, based on scope type/manufacturer.
- Single Use Biopsy Valves, Y-OPSY Biopsy Valve Irrigators, Air/Water Valves, and Suction Valves and Cleaning Adapters should not be refurbished, reprocessed, or resterilized.
- Single Use Biopsy Valves, Y-OPSY Biopsy Valve Irrigators, Air/Water Valves, and Suction Valves and Cleaning Adapters are for single use only. **Reuse poses a risk of cross contamination of bodily fluids. Discard after each procedure.**
- The Single Use Cleaning Adapter should only be used to pre-clean the endoscope's air/water channel. Do NOT use during patient procedures.
- Discard biohazard valves in accordance with local, state, and federal regulations.
- NOTE: If package is damaged or opened prior to intended use, Single Use Biopsy Valves, Y-OPSY Biopsy Valve Irrigators, Air/Water Valves, and Suction Valves and Cleaning Adapters must be discarded immediately.
- Sterility is not guaranteed if package has been opened or damaged. Do not use if packaging or product is damaged.
- Contact customer service for assistance.
- Federal (USA) law restricts the sale of this device by or on the order of a physician.

RX ONLY

WARNING:

Always wear gloves and other appropriate personal protective equipment while handling such as gown and face mask. After use, these products may be potential biohazards. Handle and dispose of in accordance with accepted medical practice and applicable local, state, and federal laws and regulations.

Instructions for Use (by DEFENDO™ Single Use Valve Kit component):

Single Use Biopsy Valves for OLYMPUS®, PENTAX® and FUJIFILM™ GI Endoscopes

1. Open the sterile peel pack and remove the Single Use Biopsy Valve.
2. Position the Single Use Biopsy Valve onto the biopsy port of the endoscope.
3. Always make sure the lid of the Single Use Biopsy Valve is closed.
4. If the diameter of the accessory is >3mm, open the lid for instrumentation.
5. At the end of the procedure, remove and discard the Single Use Biopsy Valve.

STERILE EO STERILIZED USING ETHYLENE OXIDE

DO NOT RESTERILIZE

SINGLE USE ONLY

PENTAX

OLYMPUS
FUJIFILM

Single Use Y-OPSY™ Valve for OLYMPUS® and FUJIFILM™ GI Endoscopes

STERILE EO STERILIZED USING ETHYLENE OXIDE

DO NOT RESTERILIZE

SINGLE USE ONLY

1. Open the sterile peel pack and remove the Single Use Y-OPSY™ Biopsy Valve Irrigator. Position the Single Use Y-OPSY Biopsy Valve Irrigator to the biopsy port of the OLYMPUS® or FUJIFILM™ Endoscope.
2. Attach the ENDOGATOR™ Disposable Irrigation Tubing to the Single Use Y-OPSY Biopsy Valve Irrigator via the backflow valve. The Single Use Y-OPSY Biopsy Valve Irrigator is only to be used in conjunction with ENDOGATOR Disposable Irrigation Tubing.
3. **CAUTION: DO NOT REMOVE THE BACKFLOW VALVE.** Backflow valves are permanently attached.
4. Slowly insert the instrument into the Single Use Y-OPSY Biopsy Valve Irrigator. If the diameter of the accessory is >3mm, open the lid for instrumentation. NOTE: Lid must be closed for irrigation, suction, and insufflation.
5. Reduce irrigation pump setting when irrigating while utilizing biopsy instrumentation. Pump setting can vary depending on the biopsy channel size and the instrument size. The more obstructed the channel is, the lower the pump setting. If suctioning is affected, remove instrument from channel.
6. **WARNING: Potential Patient Safety Risk:** If the Single Use Y-OPSY Biopsy Valve Irrigator is leaking, the attached ENDOGATOR Disposable Irrigation Tubing is potentially contaminated and must be discarded to ensure patient safety.
7. At the end of the procedure, remove and discard the Single Use Y-OPSY Biopsy Valve Irrigator and the ENDOGATOR Disposable Irrigation Tubing.

Single Use Air/Water Valve for OLYMPUS, PENTAX® 90 Series and FUJIFILM 500/600 Series GI Endoscopes

STERILE EO STERILIZED USING ETHYLENE OXIDE

DO NOT RESTERILIZE

SINGLE USE ONLY

1. Open the sterile peel pack and remove the Single Use Air/Water Valve.
2. Push down on the Single Use Air/Water Valve, twisting slightly back and forth, until the valve engages on the air/water port.
3. Prior to the procedure, depress Single Use Air/Water Valve to prime air/water channel.
4. Discard the Single Use Air/Water Valve after each procedure prior to reprocessing the endoscope.

OLYMPUS

PENTAX

FUJIFILM

Single Use Suction Valve for OLYMPUS, PENTAX 90 Series and FUJIFILM 500/600 Series GI Endoscopes

STERILE EO STERILIZED USING ETHYLENE OXIDE

DO NOT RESTERILIZE

SINGLE USE ONLY

1. Open the sterile peel pack and remove the Single Use Suction Valve.
2. Push down on the Single Use Suction Valve, twisting slightly back and forth, until the valve engages on the suction port.
3. Prior to the procedure, depress the Single Use Suction Valve.
4. Discard the Single Use Suction Valve after each procedure prior to reprocessing the endoscope.

OLYMPUS

PENTAX

FUJIFILM

Single Use ENDOGATOR™ Connector

STERILE EO STERILIZED USING ETHYLENE OXIDE

DO NOT RESTERILIZE

SINGLE USE ONLY

1. Open the sterile peel pack and remove the Single Use ENDOGATOR™ Connector. Attach the Single Use ENDOGATOR Connector to the GI endoscope's auxiliary water port by engaging the threads and turning slowly until you feel resistance.
2. Firmly attach to the ENDOGATOR Disposable Irrigation Tubing backflow valve via the luer lock connection.
3. Prior to the procedure, activate the footswitch and prime the ENDOGATOR Disposable Irrigation Tubing and GI endoscope with sterile water.
4. Discard the Single Use ENDOGATOR Connector after each procedure.

OLYMPUS

PENTAX

FUJIFILM

**WARNING:****Single Use Cleaning Adapter:**

Only use to pre-clean the endoscope's air/water channel. Do NOT use during patient procedures, this will result in continuous insufflation and patient injury. DO NOT REMOVE WARNING TAG.

1. After the patient procedure is complete, turn off the insufflation/airflow system and remove the Air/Water Valve from the endoscope.
2. Open the sterile peel pack and remove the Single Use Cleaning Adapter.
3. Attach the Single Use Cleaning Adapter to the endoscope's air/water port. Push down on the adapter, twisting slightly back and forth until the adapter engages on the air/water port.
4. Turn on the insufflation/airflow system and set it to maximum air flow setting.
5. Depress the button on the Single Use Cleaning Adapter for 30 seconds to flush the air/water channel with water.
6. Release the button to purge the air/water channel with air for 10 seconds.
7. Discard the Single Use Cleaning Adapter after each use prior to reprocessing the endoscope.

DEFENDO™ Single Use Valve Kits have been internally qualified by Medivators and are considered compatible single use replacement accessories for OLYMPUS® and FUJIFILM™ GI Endoscopes (included in kits 100301 and 100303), PENTAX® GI Endoscopes (included in kit 100302), OLYMPUS GI Endoscopes (140/160/180/190 Series) (included in kits 100305, 100306, 100307, 100310, 100311, 100322, and 100323), FUJIFILM 500/600 Series Endoscopes (included in kits 100312, 100313, and 100314), PENTAX 90 Series GI Endoscopes (included in kits 100315, 100316, and 100317).

DEFENDO™, ENDOGATOR™ and Y-OPSY™ are trademarks of Medivators Inc.

All other company or product names referenced in this document are trademarks of their respective owner.

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Cantel Medical (Italy) S.r.l.
Cantel (Australia) Pty Ltd